

Multi-use Syringe Care



Extension
UNIVERSITY OF WISCONSIN-MADISON

Bill Halfman, Beef Outreach Specialist
Extension Educators Sandy Stuttgen, Carolyn Ihde, and Adam Hartfiel

Cleaned

Heat
Sanitized

Stored

Cleaned Immediately After Use

- All internal components, tubing, vent spikes
- Taken apart if possible or repeated rinsing
- Use ONLY hot PURIFIED water – no soap, no disinfectants



Heat to Sanitize Clean Syringes

Options include:

- Boiling
- Microwave



**Store Clean Sanitized Syringes in
Sealed Plastic Bags in the Freezer**

Citations:

- Beef Quality Assurance National Manual. 2019 Cattlemen's Beef Board and National Cattlemen's Beef Association.
- Animal Health Equipment Management. Veterinary Clinics of North America: Food Animal Practice. Vol 31, Issue 2, July 2015, pages 259-267.
<https://doi.org/10.1016/j.cvfa.2015.03.009>

EEO/AA employer, University of Wisconsin-Madison Division of Extension provides equal opportunities in employment and programming, including Title VI, Title IX, the Americans with Disabilities Act (ADA) and Section 504 of the Rehabilitation Act requirements.